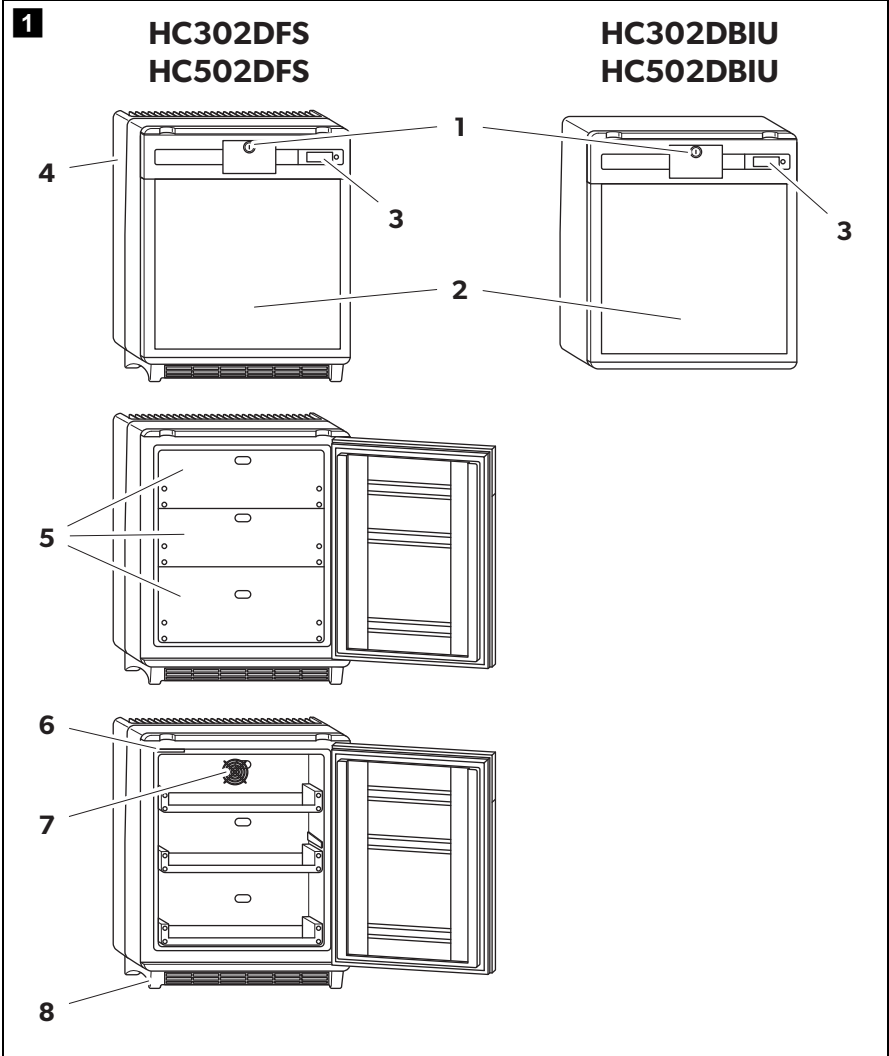
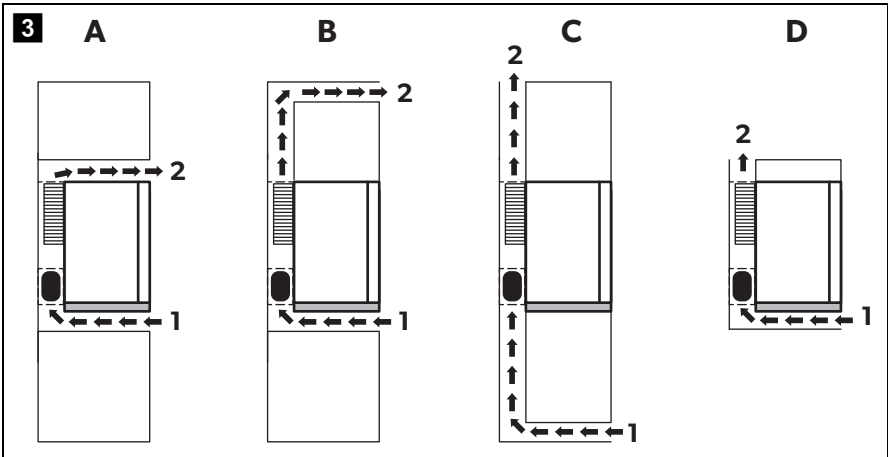
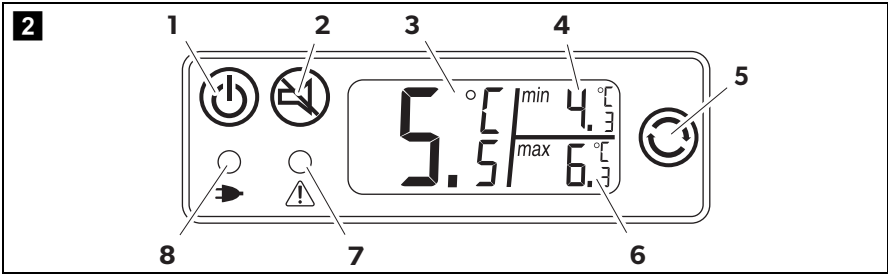
PROFESSIONAL COOLING PHARMACEUTICAL

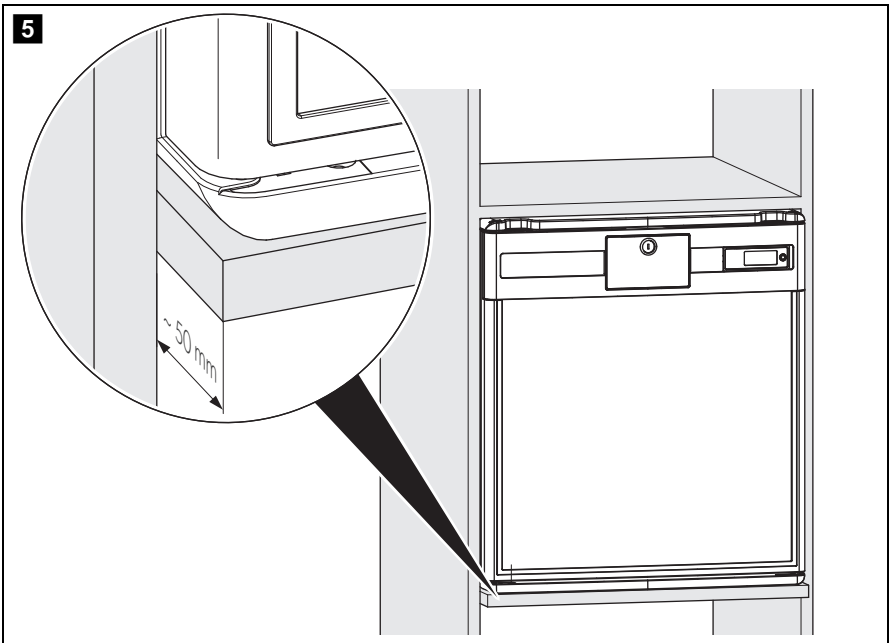
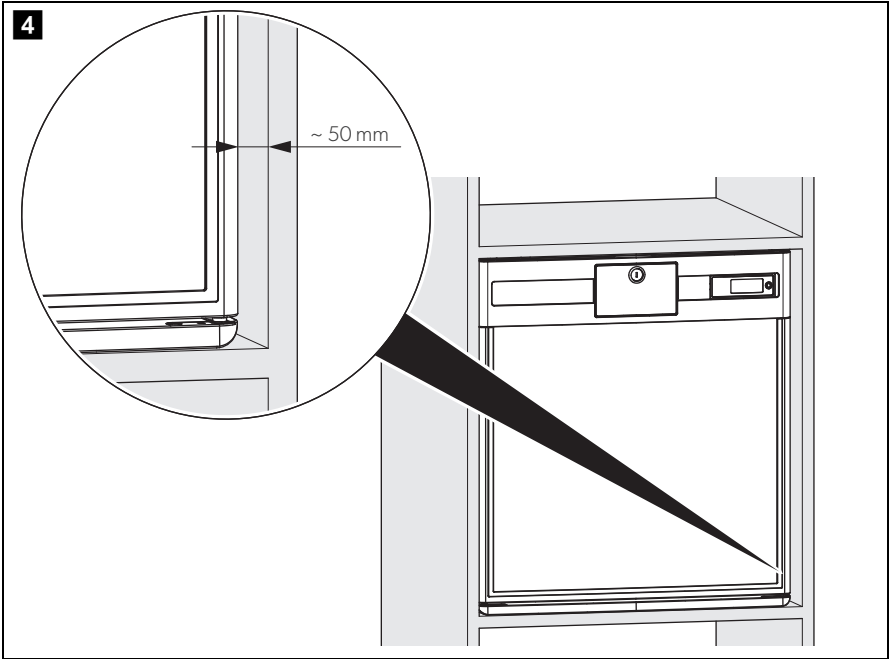
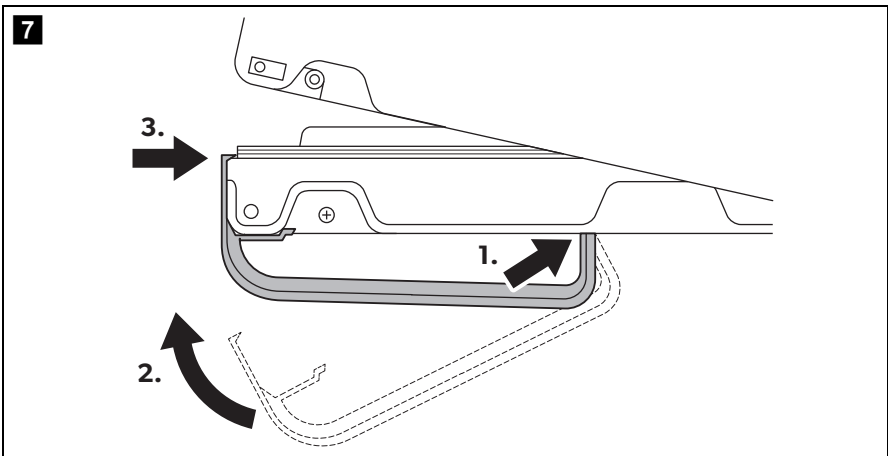
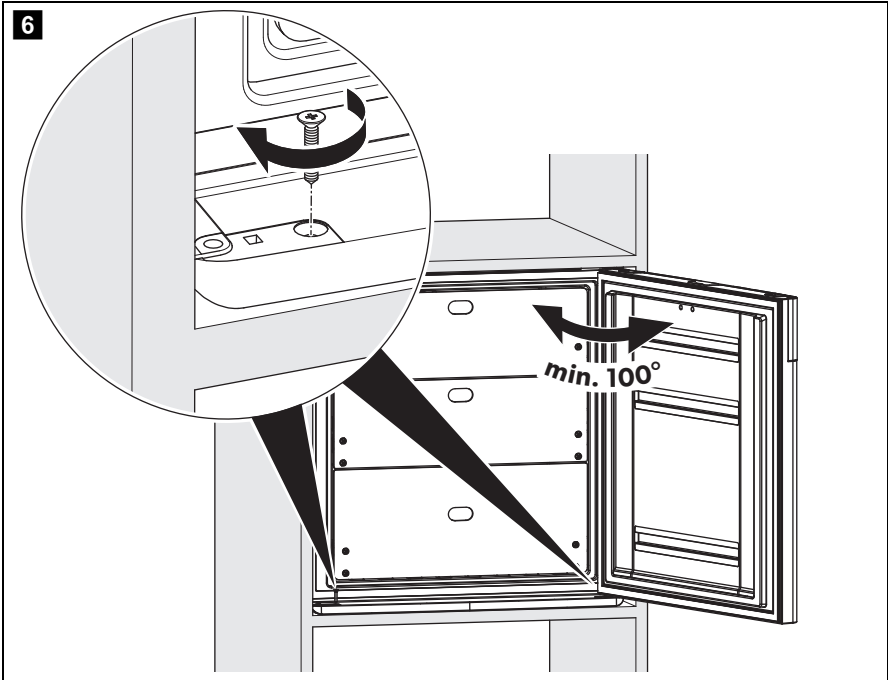
HC302D, HC502D

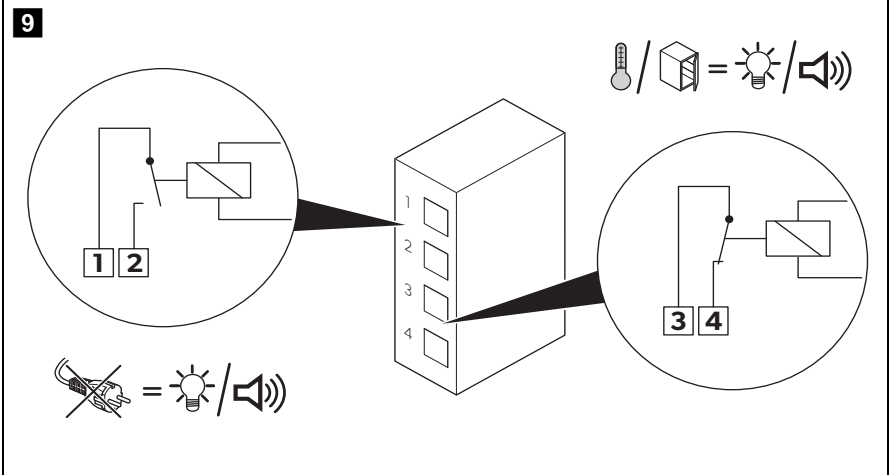
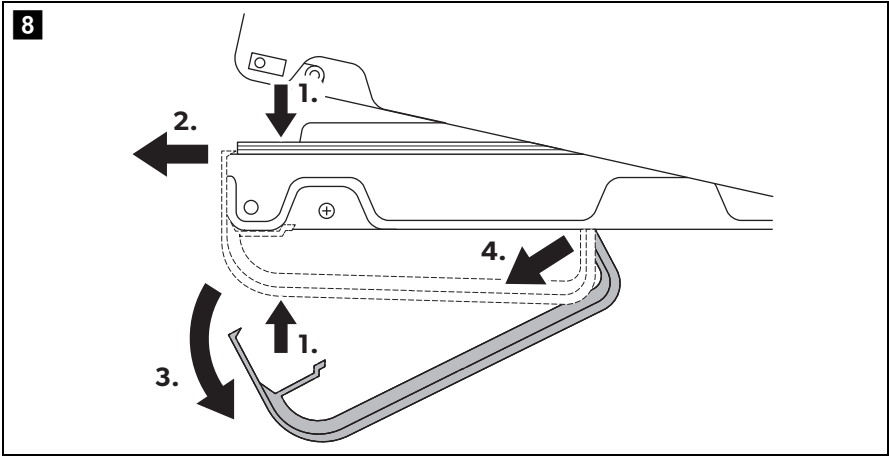
EN	Health care refrigerator Installation and Operating Manual 10
DE	Kühlschrank für medizinische Produkte Montage- und Bedienungsanleitung 29
FR	Réfrigérateur de soins de santé Instructions de montage et de service 47
NL	Medische koelkast Montagehandleiding en gebruiksaanwijzing 66

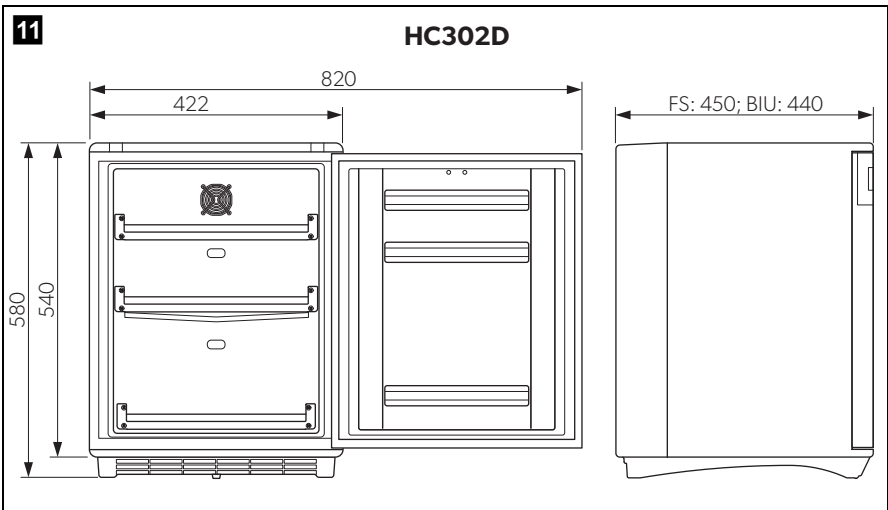
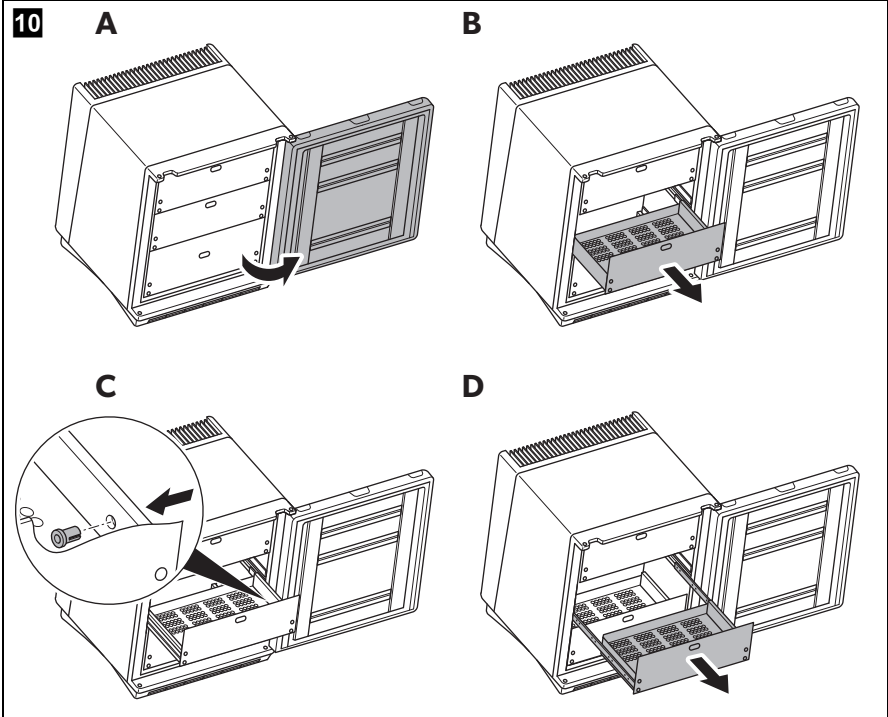
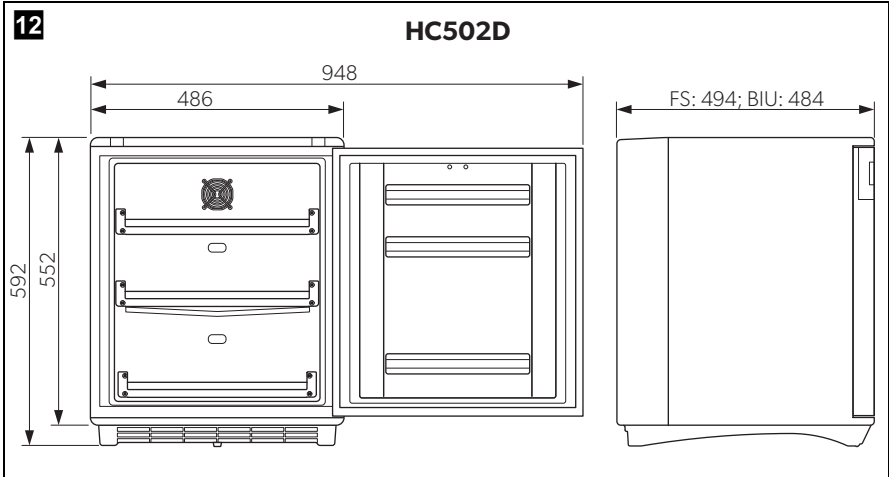
Please read this instruction manual carefully before installation and first use, and store it in a safe place. If you pass on the product to another person, hand over this instruction manual along with it.

Table of contents

1	Explanation of symbols	11
2	Safety instructions	11
3	Scope of delivery	13
4	Intended use	14
5	Technical description	14
6	Installing and connecting the refrigerator	17
7	Using the refrigerator	20
8	Cleaning and maintenance.	25
9	Troubleshooting	26
10	Warranty	27
11	Disposal	27
12	Technical data	28

1 Explanation of symbols

**DANGER!**

Safety instruction: Failure to observe this instruction will cause fatal or serious injury.

**WARNING!**

Safety instruction: Failure to observe this instruction can cause fatal or serious injury.

**CAUTION!**

Safety instruction: Failure to observe this instruction can lead to injury.

**NOTICE!**

Failure to observe this instruction can cause material damage and impair the function of the product.

**NOTE**

Supplementary information for operating the product.

2 Safety instructions

The manufacturer accepts no liability for damage in the following cases:

- Faulty assembly or connection
- Damage to the product resulting from mechanical influences and excess voltage
- Alterations to the product without express permission from the manufacturer
- Use for purposes other than those described in the operating manual

2.1 General safety

WARNING!

- Do not operate the device if it is visibly damaged.
- If this device's power cable is damaged, it must be replaced by customer service or a similarly qualified person in order to prevent safety hazards.
- This device should be repaired by qualified person only. Inadequate repairs may cause serious hazards.
- This device can be used by children aged 8 years or over, as well as by persons with diminished physical, sensory or mental capacities or a lack of experience and knowledge, providing they are supervised, or have been taught how to use the device safely and are aware of the resulting risks.
- Cleaning and user maintenance must not be carried out by children without supervision.
- Children must be supervised to ensure that they do not play with the device.
- Always keep and use the device out of the reach of children under 8 years of age.
- Do not store any explosive substances, such as spray cans with propellants, in the device.

NOTICE!

- Check that the voltage specification on the type plate is the same as that of the power supply.
- Never pull the plug out of the socket by the connection cable.
- The refrigerator is not suitable for storing substances which are caustic or contain solvents.
- Keep the drainage outlet clean at all times.
- Do not open the refrigerant circuit under any circumstances.
- Never transport the device in a horizontal position, so that no oil can leak out of the compressor.
- Make sure that the refrigerant circuit is not damaged during transportation.
- Set up the device in a dry location where it is protected against splashing water.

2.2 Operating the device safely

DANGER!

- Do not touch exposed cables.

CAUTION!

- Before starting the device, ensure that the power supply line and the plug are dry.

NOTICE!

- Do not use electrical devices inside the cooling device unless they are recommended by the manufacturer for that purpose.
- Do not place it near naked flames or other heat sources (heaters, direct sunlight, gas ovens etc.)
- **Danger of overheating!**
Always ensure sufficient ventilation so that the heat generated during operation can dissipate. Make sure that the device is sufficiently far away from walls and other objects so that the air can circulate (fig. **3**, page 4).
- Ensure that the ventilation vents are not covered.
- The device shall not be exposed to rain.
- Never immerse the appliance in water.
- Protect the appliance and power cable against heat and moisture.
- Make sure that the stored substances do not touch the walls of the cooling area.

3 Scope of delivery

Quantity	Description
1	Refrigerator
1	Handle
1	Installation and operating manual

4 Intended use

The refrigerator is designed to store medicines and medicinal products (according to DIN 58345) which should be stored at temperatures between +2 °C and +8 °C.

CAUTION!

The refrigerator is not suitable for the storage of blood.

5 Technical description

5.1 Function description

All materials used in the refrigerator are compatible for use with medicines and medicinal products.

The refrigerant circuit is maintenance-free.

The refrigerator is available as freestanding version (FS) and as built-in version (BIU).

The freestanding version is intended for freestanding use. The cooling unit cover protects the warm parts of the cooling units. The foot frame with ventilation openings ensure sufficient ventilation. By removing the foot frame and cooling unit cover the refrigerator can be used as a built-in version.

The built-in version is intended to be mounted into furniture. It doesn't have a foot frame or cooling unit cover.

The digital thermometer shows and stores the minimum and maximum inner temperatures from the last reset so that you can check whether the medicinal products have been stored at the appropriate temperatures.

5.2 Control elements

No. in fig. 1 , page 3	Explanation
1	Door lock
2	Door
3	Digital thermometer (see fig. 2 , page 4)
4	Cooling unit cover
5	Drawers
6	Interior light
7	Fan
8	Foot frame

Key to fig. **2**, page 4

No.	Explanation
1	Power button 
2	Mute button 
3	Current inner temperature
4	Minimum inner temperature
5	Reset button 
6	Maximum inner temperature
7	LED (red) for alarms
8	LED (green) for power presence

5.3 Safety thermostat

The device is fitted with a safety thermostat. This thermostat switches off the compressor as soon as the inside temperature falls below +2 °C preventing the medicinal products from being harmed by freezing.

5.4 Rechargeable battery

In case the power supply is interrupted, the monitoring functions of the electronics are maintained by an integrated rechargeable battery for at least 12 hours.

The battery is automatically charged if the device is connected to the mains and switched on.

At the first starting up of the device the error message "E3" may appear. The message will disappear as soon as the battery is charged.

If the capacity of the battery is no longer sufficient to take over the monitoring functions the error message "E3" appears on the display.

NOTICE!

The battery should be precautionary replaced every 2 years. This replacement must be carried out by qualified personnel.

5.5 Internal ventilation

The device is fitted with an internal ventilation.

The fan is operating when the door is closed.

NOTICE!

The airflow to the fan must never be impeded by blocking or covering the ventilation slots.

To prevent warm outside air from being unnecessarily sucked in, the fan automatically switches off when the door is opened.

5.6 External alarm function

There are two terminals with two floating contacts on the back of the device that can be used to trigger an additional external alarm (acoustical or visual).

6 Installing and connecting the refrigerator

6.1 Unpacking the refrigerator

- Check that the packaging is not damaged.
- Remove the cardboard packaging.
- Check that the refrigerator is not damaged.

If you find damage on the packaging or at the refrigerator resulting from transport, report it immediately to the transportation firm.

6.2 Installing freestanding version

Observe the following instructions when installing:

- Install the refrigerator so that the warm air produced can easily flow away, either upwards or to the sides.
- Place the refrigerator at the chosen place.
- Leave the refrigerator to stand for at least four hours before you connect it to the power supply so that the refrigerant can flow back into the compressor.

6.3 Installing built-in version

CAUTION!

Ensure the shelf is adequate to carry the total weight of the refrigerator and the medicinal products to be stored.

Observe the following instructions when installing:

- Install the refrigerator so that the warm air produced can easily flow away (either upwards or to the front, fig. **3**, page 4).

Key to fig. **3**, page 4

No.	Explanation
1	Cold intake air
2	Hot waste air

- Make sure that the refrigerator door is able to open at least 100°. Otherwise the drawers cannot be used.
- Check the dimensions of the device (fig. **11**, page 8 or fig. **12**, page 9).
- Place the device according to fig. **4**, page 5 or fig. **5**, page 5.
- Mount the device to the furniture by two screws through the fittings at the bottom (fig. **6**, page 6).
- Leave the device to stand for at least four hours before you connect it to the power supply so that the refrigerant can flow back into the compressor.

6.4 Connecting the refrigerator to the power supply

NOTICE!

- Connect the device to a separate circuit to prevent malfunctioning due to problems with other electrical equipment. Never connect the device to a single socket with other electrical equipment by means of a multi-socket.
- **UK only:** The refrigerator is supplied with a plug fitted with 3 A fuse. In case this fuse has to be changed, a 3 A ASTA approved (BS 1362) fuse must be used.

➤ Connect the plug of the connection cable into the mains.

✓ The green power LED lights up (fig. **2**, page 4).

➤ Press the  button.

✓ The display turns on.

For the first two seconds the display will not show temperature values; it shows "--.-". Then the temperature read from the sensor is shown.

6.5 Mounting and removing the door handle

➤ Mount the handle as shown (fig. **7**, page 6).

➤ Remove the handle as shown (fig. **8**, page 7).

6.6 Installing an external alarm

Use the floating contacts at the back of the device as follows:

- The upper contacts correspond to the power fail alarm (fig. **9** 1 and 2, page 7).
- The lower contacts correspond to the temperature alarm (fig. **9** 3 and 4, page 7).

A voltage between 12 V and 250 V can be connected at the contacts. The maximum load must not exceed 8 A. The minimum power rating is 100 mA/5 V.

NOTE

The external alarm signal can only be switched off by eliminating the cause of the alarm.

7 Using the refrigerator

NOTE

- Before starting your new refrigerator for the first time, you should clean it inside and outside with a damp cloth for hygienic reasons (please refer to the chapter “Cleaning and maintenance” on page 25).
- Always leave some space between the medicinal products in the refrigerator to ensure efficient and even cooling. If the drawers are packed too tightly there may be unacceptable temperature values.
- Keep these operating instructions ready at hand and leave them with the device, so that all users can find out about the functions and safety regulations.

7.1 Energy saving tips

- Choose a well ventilated location which is protected from direct sunlight.
- Do not open the refrigerator more often than necessary.
- Do not leave the doors open for longer than necessary.
- Defrost your refrigerator as soon as a layer of ice forms.

7.2 Starting up the device

CAUTION!

Do not store medicines and medicinal products before the E6 error code has disappeared and the operating temperature range has been reached.

NOTE

The temperature values are shown in °C. This cannot be changed.

- At the first starting up of the device or after switching it off for a long time the display of the digital thermometer (fig. **2**, page 4) shows the current ambient temperature alternating with the error message "E6".
 - The alarm LED will light up but no audible alarm will sound.
 - As soon as the inner temperature reaches the operating temperature range (2 °C – 8 °C) the current temperature is shown.
- Reset the thermometer to avoid a wrong maximum temperature value.
- Press the  button for 5 seconds.

7.3 Display backlight in battery mode

The thermometer display has a blue backlight to make the temperature values easier to read. The display backlight works continuously while the device is connected to the mains.

In battery mode the display backlight will switch off after 2 minutes.

- To activate the display backlight for 3 seconds, briefly press the  button.

7.4 Temperature values

The thermometer stores the values of the minimum and maximum inner temperature since the last reset.

To reset these values:

- Press the  button for 5 seconds.

7.5 Alarms

NOTICE!

- If you are not able to eliminate the problem triggering the alarm, store the medicinal products somewhere else as soon as possible.
- In case of power failure, the inner temperature will raise in 2 hours (hold over time) from 2 °C to 10 °C at an ambient temperature of 25 °C.

There are five special cases when alarms occur:

Alarm event	Number of beeps	Duration of the beeps
Temperature out of range	1 beep / 5 sec	1 sec
Power failure	1 beep / 3 sec	0.5 sec
Inner fan is blocked	n.a.	permanent beeping
Door open for > 20 sec		
Connection problem between display and controller	1 beep / 2 sec	1 sec
Door open for > 60 sec	2 beeps / 2 sec	0,5 sec

The alarm can be muted and unmuted:

- Press the  button.

NOTE

- If another alarm is triggered the alarm signal is automatically unmuted.
- After all alarm events have passed, a muted alarm signal must be manually reset to unmuted.

7.6 Defrosting the refrigerator

NOTE

In high ambient temperatures, the refrigerator may operate continuously, which can lead to excessive formation of frost on the rear wall of the refrigerator compartment.

Humidity can cause frost to form in the interior of the refrigerator or on the vaporiser.

A slight layer of frost or water droplets could form in the interior of the refrigerator if it has been cooling for longer periods. This is normal because the moisture in the air condenses to water when the temperature in the refrigerator drops. The refrigerator is **not** defective.

The condensate flows out through a water drain duct into a water container at the back of the device at top of the compressor, where it evaporates.

It is recommended to defrost the refrigerator every three weeks.

Proceed as follows:

- Switch off the refrigerator: press the  button for more than 5 seconds.
- Disconnect the refrigerator from the mains.
- Remove the drawers (see chapter “Removing the drawers” on page 24).
- Leave the door open.
- After defrosting wipe out the cabinet interior with a clean, damp cloth.
- Assemble the drawers.

7.7 Switching off and storing the refrigerator

If you do not intend to use the refrigerator for a long time, proceed as follows:

- Switch off the refrigerator: press the  button for more than 5 seconds.
- Disconnect the refrigerator from the mains.
- Clean the refrigerator (see chapter “Cleaning and maintenance” on page 25).
- Leave the door slightly open.
This prevents smells from building up.

7.8 Removing the drawers

You can remove the drawers, e.g for cleaning.

Proceed as follows (fig. **10**, page 8):

- Open the door (**A**).
- Pull out the drawer (**B**).
- Push the fixing pins from the outside to the inside (**C**).
- Remove the drawer (**D**).
- Assemble the drawers in reverse order.

8 Cleaning and maintenance

WARNING!

Always disconnect the refrigerator from the mains before you clean and service it.

NOTICE!

- Do not use abrasive cleaning agents or hard objects during cleaning as these can damage the refrigerator.
- Never use hard or sharp tools to remove ice or to free objects frozen onto the device.
- Do not use any mechanical tools or any other tools to speed up the defrosting process.

- Clean the refrigerator regularly and as soon as it becomes dirty with a damp cloth.
- Make sure that no water drips into the seals. This can damage the electronics.
- Wipe the refrigerator dry with a cloth after cleaning.
- Check the condensate drain regularly.
Clean the condensate drain when necessary. If it is blocked the condensate collects on the bottom of the refrigerator.
- Clean the door seals regularly.
- **Freestanding version:** Clean the grid of the condenser and the compressor on the back of the device with a vacuum cleaner regularly.

9 Troubleshooting

Water in or under the refrigerator

Fault	Possible cause	Remedy
Water is leaking into the inside of the refrigerator.	The condensate drain duct is clogged.	Clean the drain.
	Medicines are preventing the water from flowing to the collector.	Make sure that medicines do not touch the rear wall.
Water is leaking onto the floor.	There is too much water in the water container.	Clean the water container.

Display error messages

Error code	Explanation
E0	Display/Control communication failed
E1	NTC1 short circuited or open connection
E2	NTC2 short circuited or open connection
E3	Battery not connected or not OK
E4	Fan not working
E5	Door open for more than 60 seconds
E6	Temperature outside range (lower 2 °C or higher 8 °C)
E7	Mains not connected
E8	Too much difference between NTC1 and NTC2 values

10 Warranty

The statutory warranty period applies. If the product is defective, please contact the manufacturer's branch in your country (see the back of the instruction manual for the addresses) or your retailer.

For repair and guarantee processing, please include the following documents when you send in the device:

- A copy of the receipt with purchasing date
- A reason for the claim or description of the fault

11 Disposal

WARNING! Children beware!

Before disposing of your old refrigerator:

- Dismantle the drawers.
- Leave the drawers in the refrigerator so that children cannot climb inside.

NOTICE!

An insulating gas is used in the device (see type plate). Check with the relevant local government authority or your local waste disposal company about proper disposal and make sure that the device and all materials are disposed of professionally.

- Place the packaging material in the appropriate recycling waste bins wherever possible.

If you wish to finally dispose of the product, ask your local recycling centre or specialist dealer for details about how to do this in accordance with the applicable disposal regulations.

12 Technical data

	HC 302D	HC 502D
Gross capacity:	33 l	49 l
Storage capacity:	22 l	35 l
Connection voltage:	220 – 240 V~	
Energy consumption:	0.381 kWh/24 h	0.423 kWh/24 h
Power consumption:	0.54 A	
Cooling capacity:	+2 °C to +8 °C	
Operation ambient range:	+10 °C to +35 °C	
Hold over time (2 °C to 10 °C) at 25 °C:	2 hours	
Refrigerant:	R134a	
Refrigerant quantity:	38 g	
CO ₂ equivalent:	0.054 t	
Global warming potential (GWP):	1430	
Dimensions:	see fig. 11 , page 8	see fig. 12 , page 9
Weight:	approx. 20 kg	approx. 22 kg
Inspection/certification:		

Contains fluorinated greenhouse gases

Hermetically sealed equipment

Complies with the following EEC directives:

- LVD directive: 2006/95/EC
- EMC directive: 2004/108/EC
- CE directive: 93/68/EC
- RoHS directive: 2002/95/EC
- WEEE directive: 2002/96/EC
- Directive DIN 58345: 2007-09

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GERMANY**Dometic WAECO International GmbH**

Hollefeldstraße 63 · D-48282 Emsdetten
 ☎ +49 (0) 2572 879-195 · 📠 +49 (0) 2572 879-322
 Mail: info@dometic-waeco.de

dometic.com**AUSTRALIA****Dometic Australia Pty. Ltd.**

1 John Duncan Court
 Varsity Lakes QLD 4227
 ☎ 1800 212121
 📠 +61 7 55076001
 Mail: sales@dometic.com.au

AUSTRIA**Dometic Austria GmbH**

Neudorferstraße 108
 A-2353 Guntramsdorf
 ☎ +43 2236 908070
 📠 +43 2236 90807060
 Mail: info@dometic.at

BENELUX**Dometic Branch Office Belgium**

Zincstraat 3
 B-1500 Halle
 ☎ +32 2 3598040
 📠 +32 2 3598050
 Mail: info@dometic.be

BRAZIL**Dometic DO Brasil LTDA**

Avenida Paulista 1754, conj. 111
 SP 01310-920 Sao Paulo
 ☎ +55 11 3251 3352
 📠 +55 11 3251 3362
 Mail: info@dometic.com.br

DENMARK**Dometic Denmark A/S**

Nordensvej 15, Taulov
 DK-7000 Fredericia
 ☎ +45 75585966
 📠 +45 75586307
 Mail: info@dometic.dk

FINLAND**Dometic Finland OY**

Mestariitie 4
 FIN-01730 Vantaa
 ☎ +358 20 7413220
 📠 +358 9 7593700
 Mail: info@dometic.fi

FRANCE**Dometic SAS**

ZA du Pré de la Dame Jeanne
 B.P. 5
 F-60128 Pleilly
 ☎ +33 3 44633525
 📠 +33 3 44633518
 Mail: vehiculesdeloisirs@dometic.fr

HONG KONG**Dometic Group Asia Pacific**

Suites 2207-11 / 22/F - Tower 1
 The Gateway - 25 Canton Road,
 Tsim Sha Tsui - Kowloon
 ☎ +852 2 4611386
 📠 +852 2 4665553
 Mail: info@waeco.com.hk

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NEW ZEALAND**Dometic New Zealand Ltd.**

PO Box 12011
 Penrose
 Auckland 1642
 ☎ +64 9 622 1490
 📠 +64 9 622 1573
 Mail: customerservices@dometic.co.nz

NORWAY**Dometic Norway AS**

Østerøyveien 46
 N-3232 Sandefjord
 ☎ +47 33428450
 📠 +47 33428459
 Mail: firmapost@dometic.no

POLAND**Dometic Poland Sp. z o.o.**

Ul. Puławska 435A
 PL-02-801 Warszawa
 ☎ +48 22 414 3200
 📠 +48 22 414 3201
 Mail: info@dometic.pl

PORTUGAL**Dometic Spain, S.L.**

Branch Office em Portugal
 Rot. de São Gonçalo nº 1 – Esc. 12
 2775-399 Carcavelos
 ☎ +351 219 244 173
 📠 +351 219 243 206
 Mail: info@dometic.pt

RUSSIA**Dometic RUS LLC**

Komsomolskaya square 6-1
 RU-107140 Moscow
 ☎ +7 495 780 79 39
 📠 +7 495 916 56 53
 Mail: info@dometic.ru

SINGAPORE**Dometic Pte Ltd**

18 Boon Lay Way 06-140 Trade Hub 21
 Singapore 609966
 ☎ +65 6795 3177
 📠 +65 6862 6620
 Mail: dometic@dometic.com.sg

SLOVAKIA**Dometic Slovakia s.r.o. Sales Office Bratislava**

Nádražná 34/A
 900 28 Ivanka pri Dunaji
 ☎/📠 +421 2 45 529 680
 Mail: bratislava@dometic.com

SOUTH AFRICA**Dometic (Pty) Ltd.****Regional Office****South Africa & Sub-Saharan Africa**

2 Avalon Road
 West Lake View Ext 11
 Modderfontein 1645
 Johannesburg
 ☎ +27 11 4504978
 📠 +27 11 4504976
 Mail: info@dometic.co.za

SPAIN**Dometic Spain S.L.**

Avda. Sierra del Guadarrama, 16
 E-28691 Villanueva de la Cañada
 Madrid
 ☎ +34 91 833 60 89
 📠 +34 900 100 245
 Mail: info@dometic.es

SWEDEN**Dometic Scandinavia AB**

Gustaf Melins gata 7
 S-42131 Västra Frölunda
 ☎ +46 31 7341100
 📠 +46 31 7341101
 Mail: info@dometicgroup.se

SWITZERLAND**Dometic Switzerland AG**

Riedackerstrasse 7a
 CH-8153 Rümlang
 ☎ +41 44 8187171
 📠 +41 44 8187191
 Mail: info@dometic.ch

UNITED ARAB EMIRATES**Dometic Middle East FZCO**

P. O. Box 17860
 S-D 6, Jebel Ali Freezone
 Dubai
 ☎ +971 4 883 3858
 📠 +971 4 883 3868
 Mail: info@dometic.ae

UNITED KINGDOM**Dometic UK Ltd.**

Dometic House, The Brewery
 Blandford St. Mary
 Dorset DT11 9LS
 ☎ +44 344 626 0133
 📠 +44 344 626 0143
 Mail: customerservices@dometic.co.uk

USA**Dometic RV Division**

1120 North Main Street
 Elkhart, IN 46515
 ☎ +1 574-264-2131